



# The LifeStyle *New Dimension* Multifocal

---

The fitting technique for The LifeStyle **New Dimension** Multifocal is the same as fitting a spherical lens. The EQ Value will be 0.2mm flatter than the Base Curve.

A well-fit lens will clear the cornea with an upper lid attachment. A lens which has been fit too flat will have minimal apical clearance with nasal/temporal decentration. A lens which has been fit too steep will have apical vaulting.

## EQ Range

7.00—8.90 in .10mm steps.

## Diameters and Distance OZ

Diameter	Std. OZ Size	Distance OZ (DOZ) can be changed in 0.25mm steps (3.0, 3.25, 3.5, 3.75, etc. up to 6.0mm)
9.0	4.0	
9.5	3.5	
10.0	3.25	
10.5	3.0	

## Fitting Exceptions

If using a 9.5 diameter, with a 3.25 DOZ, fit 0.10 steeper.

If using a 9.5 diameter, with a 3.0 DOZ, fit 0.20 steeper.

If using a 9.0 diameter, with a 4.0 DOZ, fit 0.10 steeper.

If using a 9.0 diameter, with a 3.75 DOZ or smaller, fit 0.20 steeper.

## Add Power

The Standard Add on The LifeStyle **New Dimension** Multifocal is +0.75D. The Total Add Powers are available up to a maximum of +4.00D in +0.25D steps.

The LifeStyle Gp Company, LLC  
7500 West Mississippi Avenue Suite E240  
Lakewood, CO 80226

2530 Trailmate Drive  
Sarasota, FL 34243

Phone: 888-379-6645  
Local: 303-462-3793  
Fax: 303-462-3836  
Consultation Email:  
smartsmarsource@aol.com